

# QUICK REFERENCE GUIDE

▶▶ Open Futudent Recording Software to activate live streaming	
	The active camera will <b>livestream without recording</b> . Position the camera over the treatment area.
	Place the wireless foot pedal in position for the person who will record the images.
▶▶ Record video and set bookmarks (2 options)	
	On-Screen - <b>click</b> the recording icon to start or stop recording. To <b>Add Bookmark</b>  during recording press the bookmark icon briefly.
	Foot pedal - press the pedal for <b>3 seconds</b> to start or stop recording. To <b>Add Bookmark</b>  during recording press the pedal briefly.
▶▶ Take still images (2 options)	
	On-screen - <b>click</b> the camera icon
	Foot pedal – <b>briefly press</b> the foot pedal
▶▶ Open Gallery	
	Click the arrows on the left side of the screen to open and close the <b>Gallery</b> .
 Videos  Photos	Select <b>Videos</b> or <b>Photos</b> in the Gallery to see case documentation.
 Gallery  Camera	Go to the <b>Gallery</b> or <b>Camera</b> livestream by pressing the buttons at the bottom-left of the screen.
▶▶ Add annotations and more bookmarks	
	Open a video by pressing the <b>play arrow</b> on the video thumbnail in the gallery.
	Move the video time elapsed bar to the desired location and press the <b>Add Bookmark</b>  icon. The bookmark appears as a line.
	Click the arrows on the right side of the video to open and close the <b>Bookmarks</b> .
	Double click a bookmark thumbnail to open the <b>annotation screen</b> to add notes.
▶▶ Save the recording or images on the clinic server or computer	
	Still images can be copied and pasted in to other imaging software or patient records with import functionality.
▶▶ Share with colleagues or patients using the Cloud or MyDentalBook	
	Send to patient. Attach treatment plans and home-care instructions.
	Send to colleague. Your colleague will receive a link to follow.
	Upload to cloud. Save your cases for future consultation.